

Dental Handpiece Lubrication Chart

Always lubricate after each patient and before sterilization.

Never use disinfectant or disinfectant wipes on handpieces as they are corrosive!

Compliments of:

Synteklubricant.com
Synthetic handpiece lubricants

&


Highspeed Handpieces

For dropper lubricant:


1. With blank bur installed, apply 3 drops into drive air line (see illustration below).
2. Attach handpiece to air supply. Cup head with paper towel, being careful not to touch bur. Run handpiece for 10 seconds, expelling lubricant onto towel. Repeat if expelled lubricant is discolored.
3. Remove bur, clean handpiece with alcohol wipe, and sterilize.

For aerosol lubricant:


1. Fit spray can with appropriate adapter nozzle for handpiece.
2. With blank bur installed, spray into handpiece for 2 seconds. Hold handpiece and nozzle together while spraying, pointing front of head away from your face.
3. Attach handpiece to air supply. Cup head with paper towel, being careful not to touch bur. Run handpiece for 10 seconds, expelling lubricant onto towel. Repeat if expelled lubricant is discolored.
4. Remove bur, clean handpiece with alcohol wipe, and sterilize.

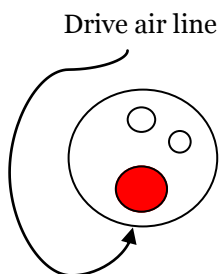


2 or 3 hole
handpiece

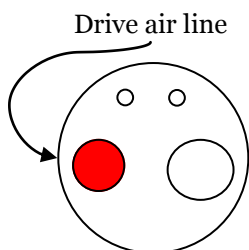

4 hole
handpiece

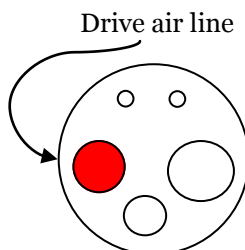

5 hole
handpiece

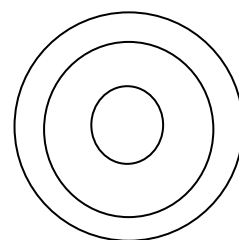

Swivel type handpiece
(use adapter nozzle)

Dental Handpiece Lubrication Chart

Always lubricate after each patient and before sterilization.

Never use disinfectant or disinfectant wipes on handpieces as they are corrosive!

Compliments of:

Synteklubricant.com

Synthetic handpiece lubricants

&

Slowspeed Handpieces

For dropper lubricant:

1. Apply 2 drops into the drive air line (see illustration). Run for 10 seconds.
2. Remove bur, clean handpiece with alcohol wipe, and sterilize.

For aerosol lubricant:

1. Fit spray can with appropriate adapter nozzle for handpiece.
2. Spray into drive air line (see illustration) for 2 seconds. Hold handpiece and nozzle together while spraying. Run for 10 seconds.
3. Remove bur, clean handpiece with alcohol wipe, and sterilize.

Electric Handpieces

1. Fit spray can with appropriate adapter nozzle for handpiece.
2. Remove handpiece from motor and bur from handpiece.
3. Slide lubrication adapter into back of handpiece. Spray for 2 seconds, holding handpiece and nozzle together and pointing front of head away from your face.
4. Attach handpiece to motor, replace the bur and run for 20 seconds to purge.
5. Repeat if expelled lubricant is discolored.
6. Remove bur, clean handpiece with alcohol wipe, and sterilize.

Attachments & Removable Heads

1. Use dropper to lubricate all moving parts & run briefly.
2. Remove bur, clean handpiece with alcohol wipe, and sterilize.